



LineSnuggler

By Nursing and Patient Care Innovations, LLC

Quantity: _____

LOT: _____

Inspected: _____

Product Description:

- The Line Snuggler sleeve is a central venous catheter *line cover* that is intended to protect, contain, and organize the underlying central line and the connection points between central lines, IV tubing, and caps.

Care:

- Washing instructions: Machine wash in hot water and dry on high heat every 3 days, when the IV tubing is changed, or when soiled.


Use:

- Discard after 1 month of continuous use
- Discard if torn, ripped, or stained
- For single patient use only
- Do not sew or attempt to repair
- For use in all ages and sizes
- Can fit 10-12 IV tubings all Y-ed in together going to 2 or 3 lumens of a central line

Instructions for Use:

1. Place the end of the sleeve with the wrap-around flap just above the bi- or trifurcation of the central line. (If bi- or trifurcation is under the dressing, position the sleeve just above the central line lumen clamps.)
2. Wrap the sleeve around the extending central line lumens and connected IV tubing.
3. Ensure Velcro™ is secured with no gaps all the way down the sleeve.
4. At the end (closest to the patent), wrap the two-sided Velcro™ around itself to secure the sleeve in place. (See FIG 1)

Note: For easier ambulation, the bottom of the sleeve may be connected to close to the top of the sleeve, by folding it up and connecting the hook and loop connectors on the side of the sleeve.

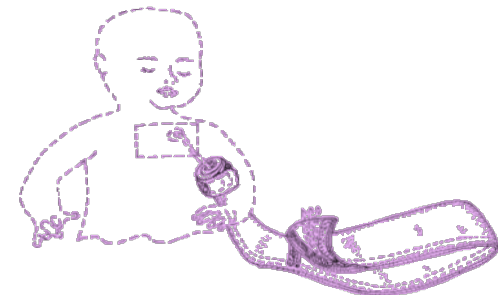

Reorder and product information:

www.LineSnugglers.com

US FDA 513(g) reviewed : Submission #C180053

